

Mobil-IZE SHOULDER

Shoulder brace with ice & compression

- Air Compression
- 2 Reusable Cooling Packs
- Universal Fit for Left/ Right Shoulder
- Targeted cold compression



TRUMED 

Please read fully before operating











Mobil-IZE SHOULDER

Shoulder brace with ice & compression

PRODUCT FEATURES:

- Lightweight and durable design
- Provides comfortable support and is easy to use
- Universal size fits both left and right shoulders
- Delivers targeted cold and compression therapy for pain relief and reduced swelling
- Includes 2 reusable gel packs, offering up to 2 hours of cold therapy

INSTRUCTIONS

-  Insert the gel ice pack, pre-cooled in the freezer for at least 2 hours, inside the shoulder brace.
-  Screw in bulb pump to the valve.
-  Feed one end of the long strap through the plastic ring at the back, following the "IN" and "OUT" markings, and then attach the fastener hook to the strap.
-  Wear the brace with the ice pack on your shoulder. Wrap the long strap around your back, thread it through the front plastic ring, and adjust it for a comfortable fit.
-  Adjust the 2 short straps around the bicep and tighten to your comfort.
-  Pump the bulb to inflate the brace to a comfortable pressure, making sure the arrow is set to "ON." Turn to "OFF" to maintain compression.
-  Close the air valve, then unscrew and remove the bulb. Attach the pump to the wrap when it is not inflating.
-  To deflate, turn to "ON" position and allow air to release.



IDEAL FOR

Provides essential support for the shoulder
Suitable for sports injuries, rehabilitation, sprains, and muscle tensions
Cold therapy helps reduce swelling, blood flow, and inflammation
Compression therapy assists in reducing blood flow and swelling



CAUTION:

1. This shoulder brace is for single patient use only.
2. It is supportive but does not guarantee injury prevention. The seller is not responsible for any injuries.
3. Consult your healthcare provider before use and follow your physician's guidance.
4. If you experience skin irritation, pain, swelling, sensation changes, or unusual reactions, consult a medical professional immediately.



STORAGE/ WORKING ENVIRONMENT

The product should be kept away from the heat source to avoid deformation and cracking. It should be stored or used at cool and dry place.

CLEANING AND MAINTENANCE

Soft fabrics can be cleaned with a damp cloth and a soft detergent to avoid acid corrosion.

WARRANTY:

The product is guaranteed to be free from defects in materials and workmanship for 180 days from purchase.

ORDER INFORMATION

1. Shoulder strap (Tru-SS105) *1 pc
2. Shoulder ice packs (Tru-CS105) *2 pcs
3. User manual (Tru-SM105 Rev A) *1 pc



401 Bloomfield Dr, West Berlin, NJ 08091 USA
855.700.4855 • support@trumedx.com • www.trumedx.com

MADE IN CHINA

©2024 Tru-SM105 Rev A